



RapidVet®-H (Feline A, B & AB) Instructions for Use

Dispense 1 drop of Diluent into the Auto-Agglutination well. Dispense 1 drop of patient blood into the same well. Stir well for 10 seconds using a stirrer.

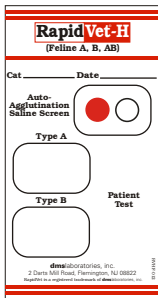


Auto-Agglutination Pre-Screen

Dispense 1 drop of Diluent into each remaining well.

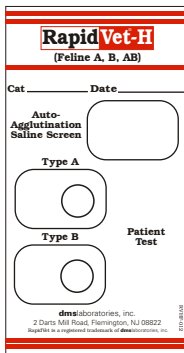


Step 1:

Dispense 1 drop of patient blood into the well marked Type A and dispense one drop of patient blood into the well marked Type B

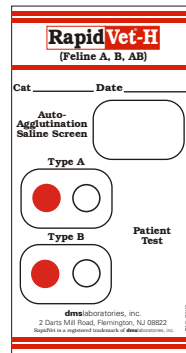

Step 2:


Stir Type A well with a stirrer for 10 seconds. Using a NEW stirrer, stir the Type B well for 10 seconds.

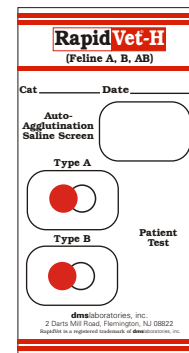


Step 3:

If no agglutination is observed in the well, then you may proceed with the test.

Add a second drop of diluent to the well marked Type A. Do not stir the well again with a stirrer.

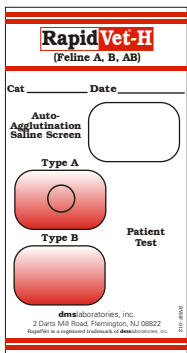


Pick the card up and rock and tilt by wrist action for 2 minutes.

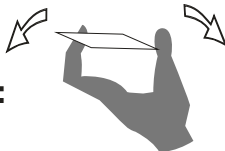

Step 5:

Position the card at a 10° - 20° angle.

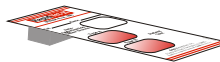

Step 6:

Read the results:

Step 7:

See Photo Interpretation Guide